

Omni Hyal

Sodium Hyaluronate

Ophthalmic Solution

Description:

Omni Hyal is clear sterile, non pyrogenic viscoelastic solution available in glass prefilled syringe.

Omni Hyal is manufactured using highly purified, high molecular weight sodium Hyaluronate derived through bio fermentation process.

Omni Hyal is free from latex.

Sodium Hyaluronate is a polysaccharide which is distributed widely in the extracellular matrix of connective tissue (vitreous and aqueous humor of the eye, synovial fluid, skin, and umbilical cord). Sodium Hyaluronate form a viscoelastic solution in water (at physiological pH and ionic strength) which makes it suitable for aqueous and vitreous humor in ophthalmic surgery.

Indications:

Omni Hyal is indicated as Ophthalmic Surgical aid –
In cataract extraction
Intraocular implantation
Corneal transplant
Glaucoma filtration and retinal attachment surgery

Contra Indications:

Hypersensitivity to Hyaluronate or any component of the formulation



Adverse events:

Postoperative inflammatory reactions (iritis, hypopyon), corneal edema, corneal decompensation, transient postoperative increase in IOP

Precautions & Warning:

Do not overfill the anterior chamber; carefully monitor intraocular pressure
Discard remaining material and do not re-sterilize the product.
Discard in case pack is open or any floater is observed in syringe.

Presentation:

Omni Hyal is available in concentration of 10 / 14 and 18 mg/ml in sterile glass prefilled syringe with filled volume 0.55/0.85/1.00 ml along with sterile, single use 27 gauge Cannula.

Storage instruction:

Store at 2-25° Centigrade
Protect from direct sun light.
Avoid freezing.



REGD. OFFICE :
OMNI LENS PVT. LTD.
5, "SAMRUDDHI", Opp. Sakar-III,
Navrangpura, Ahmedabad - 380 014. (INDIA)
Phone : +91-79-27544031-32-33-34 Fax : +91-79-27544035
E-mail : info@omnilens.in, info@bgpgroup.com
Website : www.omnilens.in

An ISO 13485 Certified Company

